




suPARnostic® Quick Test Reader - Quick Guide

How to use the reader without a PC.


This guide will show you the workflow for how to use the reader without a PC. With this guide you will be able to perform the suPARnostic® Quick Triage device measurement, retrieve an old result and to check the calibration in case of an error message.

Menu Screen No.

0.1

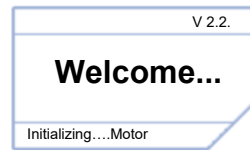

- Press and hold Enter (5) to turn on the reader

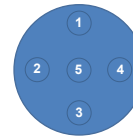


0.2



How to navigate within the menu:


- Move up and down in the menu's with (1) and (3) to select the item you want to change eg. Sample ID.
- Enter the item by pressing Enter (5).
- When the item that needs to be changed is chosen press Up (1) and Down (3) to change it.
- Exit the selected item by pressing Enter (5) again.
- If more than one page in the menu move between the pages with (2) and (4).

1.0 - Scan Menu

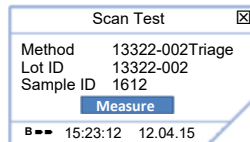


- Choose Method for the specific batch
- Display LOT ID for the current Method
- Display Sample ID
- Press 'Measure' to start measuring the device

1.1

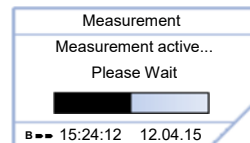

- Wait while measurement is active


1.2

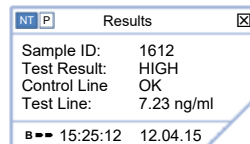

- Press NT if a new test is run
- Press P to print the result (ONLY with external printer)
- Select the cross to enter the MAIN MENU.
- Use the keys (2) and (4) to move left and right.




0.3 - Main Menu

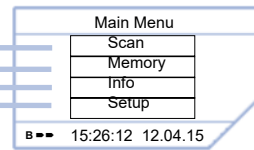


- Press on any page to get to Main Menu screen
- Choose **SCAN** to perform measurements (1.0)
- Retrieve stored results in **MEMORY** menu (2.0)
- **INFO** menu displays ID, Serial no. and firmware of the reader. *Note: Skip page 1. Use (4) to go to page 2.*
- **SETUP** menu: Time and time format configuration, reader calibration and other settings.

1.0


2.0 - Result Menu

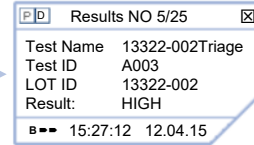

2.1

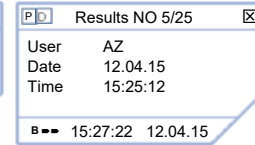


2.2

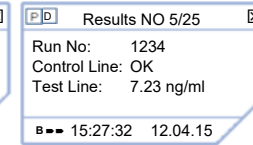


Every Result has 3 pages

- P — Print (Ext. printer needed)
- D — Delete Result

3.0 - Info Menu

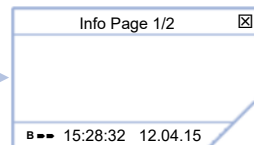


3.1

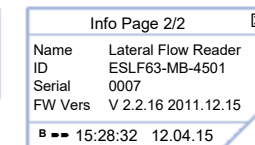


- Set via Lateral Flow Manage Software

The Menu 3.1 information will be necessary when contacting ViroGates in case of troubleshooting.


4.0 - Setup Menu

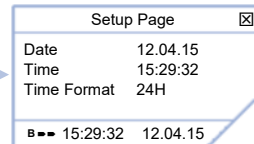

4.1

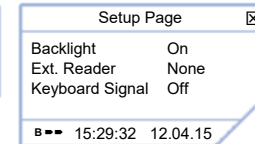

4.2

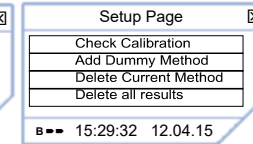

- The Setup Menu is intended for ViroGates. In case the reader reports a calibration error use the option '**Check Calibration**' shown in 4.2

ViroGates A/S
 Blokken 45
 Birkerød 3460 Denmark
 Tel: +45 2113 1336
 Skype: Virogates